



0476



## Closed anoscope with window for sutured anopexy of haemorrhoids.

REF A.4088

**L-Bet88** is an instrument allowing the proctologist to perform a suture anopexy of haemorrhoids both outpatient and in the operating room.

It is a closed tip surgical anoscope of approximately 80mm length. Like all other Sapi Med anoscopes and retractors, the hollow handle contains an optical bar for lighting. The instrument's body is cylindrical and is connected to the handle by a flange that contains a small slit on the left side through which to pass the tip of the thread therefore preventing it from making contact with the surgical zone.

The right side of the body has an elliptical window (approx. 20 mm) near the tip through which surgery can be performed on the mucous tissue that protrudes within the anoscope; the window is approx. 35 mm away from the flange in order to protect the pectinate line from the wall of the anoscope. The left side has a centimetre scale readable from the inside of the instrument. The anoscope has a closed tip which substitutes the obturator during its introduction and allows to keep the operating area clean from any faecal residuals present in the ampulla. The adhesive on the flange lets the surgeon operate independently.

*Available in version:*

**REF A.4088** anoscope and knot-pusher. Self-retaining, single-use and sterile.

**REF A.4088.4** anoscope, knot-pusher and 4 Vicryl 2/0 suture threads and 5/8 needle of 26mm. Self-retaining, single-use and sterile.